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Instructions for Use

What are Tongue Tamers® PLUS?

- 9 reminder protrusions with rounded ends for tongue training and 6 undercuts for crossbite elastics on a bondable button type pad with 80-gauge mesh pad (Fig.1).

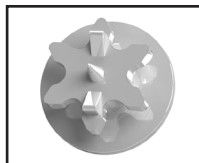
Fig.1

Why use Tongue Tamers® PLUS?

- Rapid Anterior and Lateral Open Bite Closure (ROC).
- Prevents anterior-lateral tongue positioning.
- In lateral open bites, undercuts are useful additionally for 1/4", 6oz heavy crossbite elastics at premolars, molars (Fig.2).

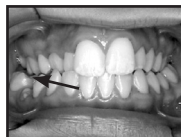
Fig.2a

Fig.2b

When are Tongue Tamers® PLUS used?

- In early and adult treatments with orthodontic brackets.
- At the second appointment after brackets have been placed.

How do Tongue Tamers® PLUS work?

- Tongue retracts from forward positioning using a negative conditioning response, similar to a hot-stove reflex of the tongue (Fig.3). Tongue Tamers® PLUS are reminders and do not cut tongue or operator's gloves or skin.

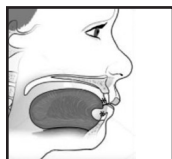
Fig.3

Where are Tongue Tamers® PLUS placed?

- Palatal of upper anteriors 3 to 3 and Lingual of lower anteriors 3 to 3, bonded just below the center of the crown to permit open bite closure (Fig.4).
- Bonded to the premolars and molars for lateral open bites.

Fig.4

Special Techniques

- As an open bite closes rapidly, Tongue Tamers® PLUS occasionally need to be trimmed with high speed bur to permit continued open bite closure (Fig.5).

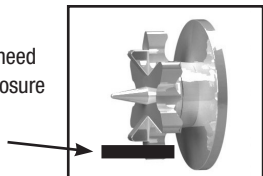
Fig.5

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